

***DenLine Protection Plus® Care Instructions for
Home Style and Laundromat Washer & Dryer 6-13-22***

About the fabric used in DenLine Protection Plus Lab Coats and Apparel...

DenLine Protection Plus has been specially engineered to protect the wearer from incidental splash of aqueous fluids (i.e., blood, body fluids, bleach, etc.), at the same time providing a high level of air porosity and comfort, stain resistance and long life.

Follow these guidelines for maximum performance and life from your coats...

Launder before wearing to Sanitize Coats: Launder Only, do not dry clean.

#1 Pretreat Spots and Stains Work ***Awesome*** or ***Dawn*** Dishwashing Soap into fabric.

#2 Laundry Detergent- Use soaps with pH of 9.0 or less for maximum life of coats.

Suggestions:

Tide PurClean (pH 8.6), Tide Simply (pH 8.8), Gain Ultra (pH 8.6),

Persil ProClean (pH 8.6).

Or Google **Detergent Name** and request **SDS Specifications** showing pH.

IMPORTANT: DO NOT USE SOFTENERS

#3 Wash Temperatures & Chemistry to Sanitize Coats:

* Hot Water Wash (160 Degrees F). No Bleach needed.

* Warm Water (<160F) Add ***Clorox Regular Bleach2 with CLOROMAX*** (6% sodium hypochlorite) to wash cycle.

10 Gallon Wash Machine- Add 1/2 Cup Bleach* Wash 10 Minutes

16 Gallon Wash Machine-Add 3/4 Cup Bleach * Wash 10 Minutes

* *Source- Clorox Healthcare™ -EPA List of Registered Disinfectant Products*

Use Machine Dispenser for injecting chemicals and soap...do not pour bleach directly on coats.

IMPORTANT: DOUBLE RINSE COLD TO REMOVE ALL SOAP AS WELL AS BLEACH IF USED.

IMPORTANT: Do NOT Drip Dry

Do NOT use DRYER SHEETS (Bounce, etc.)

#4- Dryer Temperature Specs

Use settings labeled **Perm Press**, **Easy Care** or **Wrinkle Free**. Let coats remain in dryer through complete cool down cycle. Removing coats while hot will set wrinkles.

Temperatures not to exceed 160F. Excessive Heat may distort and damage cuffs and fabric. Consult a Service Technician if you experience what appears excessive heat.

#5- If Pressing is desired, use POLYESTER SETTING Only. When turning iron on, allow iron to preheat before ironing. A Hot Iron will distort, scorch or melt fabric.

To brighten White Cuffs, soak in a 50/50 solution of bleach & water for 20 minutes, squeeze through, rinse thoroughly ..then immediately launder garment. Do not use this process on colored cuffs.

#6- DenLine Coats may be Autoclaved. Place loosely in a Sterilization Pouch- 274 F/ 134 C for 4 Minutes.

Any Questions? Call DenLine Uniforms at 800-336-5463 or 217-228-9272